

ELECTRODE CARE

THE DO'S AND DON'TS FOR PRODUCT LONGEVITY


WITH PROPER MAINTENANCE YOUR ELECTRODES WILL PERFORM BETTER AND LAST LONGER.

Please note that when electrodes show wear and tear they should be replaced with a new set to ensure that you will continue to receive quality performance. Remember, Electrodes are disposable.


THE FOLLOWING GUIDELINES WILL HELP YOU GET MORE USE OF YOUR ELECTRODES.




Make sure skin is clean and free from grease and lotion.

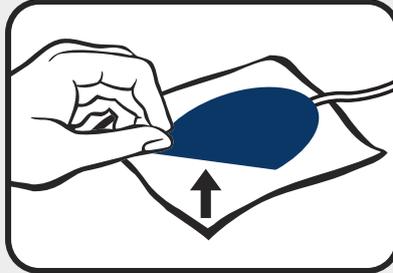


Remove Electrodes by lifting the edge.

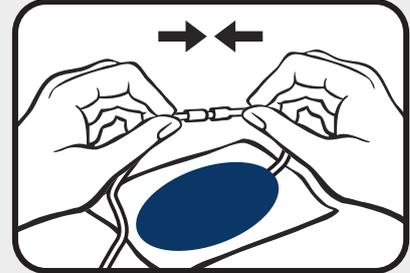

While disconnecting or inserting pin, hold the pin connector

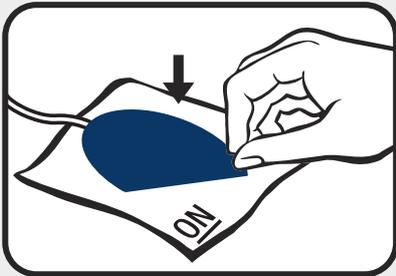


Return Electrodes to the «ON» side of the Release Liner.

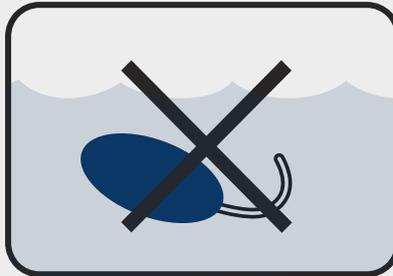


Do not submerge Electrodes in water.



Do not remove Electrodes by pulling the leadwire.

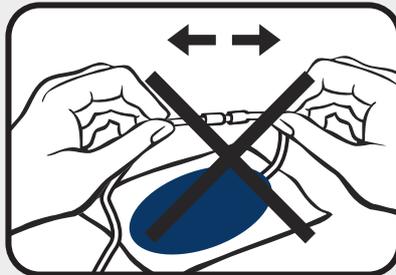

Do not pull the leadwire to remove the pin.

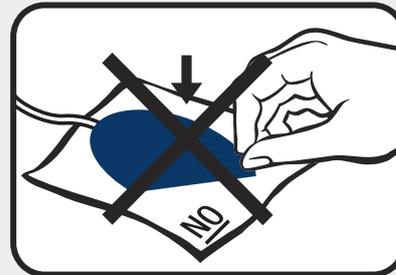

Do not place Electrodes on the «NO» side of the Release Liner.

Additional Tips :

- Electrodes are intended for single patient use.
- Consult electrodes packaging and stimulator manual for proper use of electrodes.
- If an Electrode's gel becomes dry, re-hydrate it with droplets of water.
- When not in use, always reseal Electrodes in their original package and store at room temperature.
- The use of hot and cold packs will cause an Electrode's gel to break down over time.

VEINOPUS®

Ad Rem Technology - Certified ISO 13485 : 2012

162, rue du Faubourg Saint Honoré - 75008 Paris, France - Tél. +33 (0) 1 42 60 00 22 -

www.veinoplus.com

created the 19/08/14